




Surgical technique MY SCREW Ø2.5

(The specific ancillary equipment is required for fitting ORTHO CAPE implants)

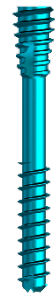

For all our cannulated screws the operating technique remains the same:

- Inserting the pins
- Spindle gauge
- Passage of the cannulated drill
- Passage of a head drill if necessary
- Tighten the screw with the appropriate screwdriver.

After creating a working chamber by detaching the soft parts of the bone using a rasp, introduction of the burr.

The choice of burr depends on the user's habits. Here a Shannon short 38.37.01



Using a spacer or rigid pin move the head of the metatarsal according to the correction sought




Pin fixation of the displaced head through the metatarsal shaft.

Insert the measurer onto the pin.

Direct reading of the screw to use.

Here a MY SCREW 28mm long

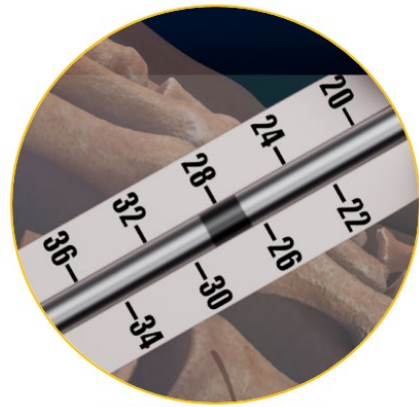

Introduction of the cannulated drill onto the pin in the head

Repeat the operation on the second pin



Screwing using the appropriate cannulated screwdriver.

The pin can be removed first.

