

OrthoCAPE

Surgical technique PPDA Ø2.6 & Ø3.0

(The specific ancillary equipment is required for fitting ORTHO CAPE implants)



For all our cannulated screws the operating technique remains the same:


- Inserting the pins
- Spindle gauge
- Passage of the cannulated drill
- Passage of a head drill if necessary
- Tighten the screw with the appropriate screwdriver.

After creating a working chamber by detaching the soft parts of the bone using a rasp, introduction of the burr.

The choice of burr depends on the user's habits. Here a Shannon short 38.37.01



Approximation of the cortices and insertion of the holding pin.

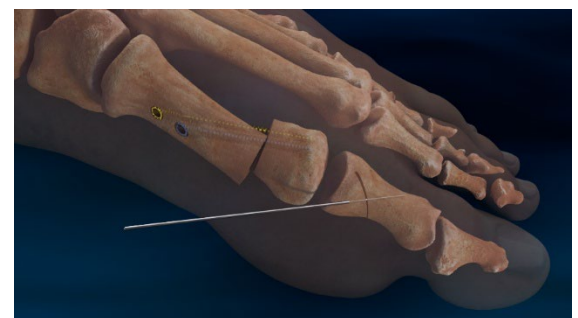

Insert the measurer onto the pin.


Direct reading of the screw to use.

Here a PPDA Ø2.6 LG 28



Introduction of the appropriate cannulated drill onto the pin in the head

Repeat the operation on the second pin

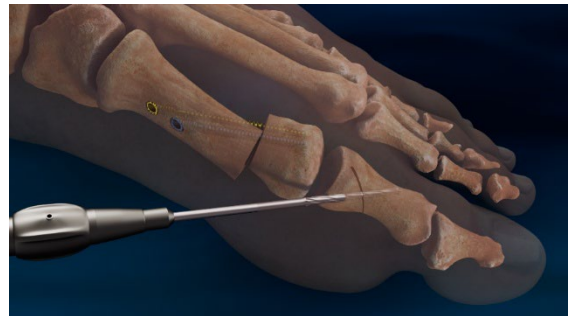

Insert the screw using the screwdriver until the head is fully inserted.

The pin can be removed first.

